



CE certified
(CE0197)



NADAL® COVID-19 Ag Test

Antigen rapid test for self-use, for the detection of an infection with SARS-CoV-2

NADAL® COVID-19 Ag Self Test Tried and tested


Our NADAL® COVID-19 Ag Self Test offers the same quality as our tests for professional users. In fact, the tests are identical in almost every way. Both the Paul Ehrlich Institute and a team at the Berlin Charité led by Prof. Drosten evaluated the test for professional users and gave it top ratings. This is how the German Federal Ministry of Health also became aware of our NADAL® COVID-19 antigen test and subsequently integrated it into their national testing strategy.

NADAL® COVID-19 Ag Self Test: A product of almost 40 years of experience

We are not newcomers, nor have we jumped on the rapid test production bandwagon since the outbreak of the pandemic. We have 39 years of experience in the production and sale of *in-vitro* diagnostics. We are specialists in what we do. Our in-house laboratory is continually pushing the boundaries of innovation. Even in the midst of a pandemic, we were able to react quickly and were one of the first companies worldwide to bring COVID-19 antigen tests to the market. The NADAL® COVID-19 Ag Self Test also stands out in this respect in terms of its quality and value.

Your benefits at a glance:

- ✓ German manufacturer with EN ISO 13485 and ISO 9001 certification
- ✓ CE certification (CE0197)
- ✓ Nasal sampling
- ✓ High specificity
- ✓ High sensitivity
- ✓ Fast and reliable results in just 15 minutes
- ✓ All test components – including sterile swabs – are included
- ✓ Reliable detection of the virus variants Alpha, Beta, Gamma, Delta, Epsilon, Theta, Iota, Kappa, Lambda, Mu and Omicron.
- ✓ Instructions for use in 17 languages

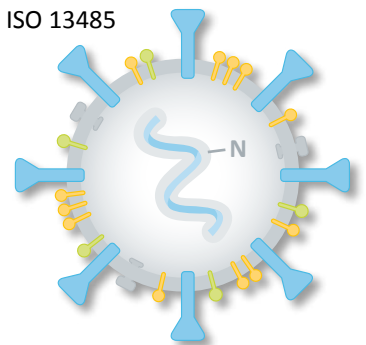

Test procedure

The diagram illustrates the 11 steps of the test procedure:

1. Prepare the test cassette.
2. Open the ampoule.
3. Add the sample to the test cassette.
4. Seal the test cassette.
5. Perform a nasal swab.
6. Perform a second nasal swab, approximately 2 cm deep.
7. Mix the sample 10 times.
8. Insert the swab into the test cassette.
9. Add the reagent.
10. Read the result after 3 minutes.
11. Wait for 15 minutes.

Result interpretation

- Negative result (⊖): One red line in the T line position.

 - Positive result (+): Two red lines, one in the C line position and one in the T line position.

 - Invalid result (⊗): No red lines or only one red line in the C line position.

A detailed description can be found in the instructions for use.

NADAL® COVID-19 Ag Test

Antigen rapid test for self-use, for the detection of an infection with SARS-CoV-2

Product	Sample material	Sensitivity	Specificity	Product code	Pack size
NADAL® COVID-19 Ag Test	Nasal swab	94.1%* *Ct range <30	> 99.9 %	243117N-01	1 test per pack
				243117N-05	5 tests per pack
				243117N-20	20 tests per pack

Kit contains test cassette • sterile swab • tube incl. dropper cap • ampoule • instructions for use • additional tube holder in packs of 5 and 20

Our sales team will be happy to answer any questions you may have on

Tel.: 0800 5892022 or by email at vertrieb@nal-vonminden.de

Further information:



We help you care!

Rapid Tests

Laboratory Diagnostics

Laboratory Service

Consulting & Service

Headquarters

nal von minden GmbH · Carl-Zeiss-Strasse 12 · 47445 Moers · Germany
 Chief Executive Officers: Sandra von Minden · Roland Meißner · Thomas Zander
 Commercial reg. Kleve · HRB 5679
 Tel.: +49 941 290 10-0 · Fax: +49 941 290 10-50

info@nal-vonminden.com
www.nal-vonminden.com