

Civil Oronasal Mask KN95



Disposable KN95 Civil Oronasal Mask*

The KN95 oronasal mask provides a physical barrier against particles, which can reduce the dispersion of larger droplets, e.g. those produced by the wearer's coughs. The KN95 oronasal mask is comfortable enough for prolonged wear and optimal for sensitive skin.

- ✓ Nose clip, earloops and non-woven fabrics
- ✓ 4-layer oronasal mask
- ✓ GB 2626-2006, type KN95
- ✓ physical filtration system:
polytetrafluoroethylene (PTFE) nanomembrane

Rapid Tests


Laboratory Diagnostics

Laboratory Service

Consulting & Service


User instructions

1




Unfold the mask. Place the open side of the mask under your chin so that the nose clip is snug against your face.

2



Carefully pull on the elastic loops with both hands. Secure the loops behind your ears and adjust them so they are comfortable.

3



Adjust nose clip with both hands. Check the fit of the mask.

KN95 Civil Oronasal Mask

This breathable four-layer mask consists of skin-friendly non-woven fabrics as well as elastic earloops and a metal nose clip for optimal fit. Masks are for private use only.



REF	Specifications	Available Test Reports	Pack Size
130003N-10	GB 2626-2006 Filtration efficiency $\geq 95\%$ * Respiratory resistance: Inhalation ≤ 350 Pa Exhalation ≤ 250 Pa	1. Filtration efficiency testing report 2. Respiratory resistance testing report	10 pcs / pack

* confirmed by a German textile institute

Our **customer service team** is on hand to answer any questions you may have by telephone, via email or in a personal consultation. For details on **your local sales office and regional free phone numbers**, visit www.nal-vonminden.com.

We help you care!

- Rapid Tests
- Laboratory Diagnostics
- Laboratory Service
- Consulting & Service