

new art laboratories



NEW! Improved version available 2018

- ✓ Extra long life through latest LED technology
- ✓ Higher precision by shielding stray light
- ✓ Automatic calibration with each test
- ✓ Status LED on reader front

nal von minden Rapid Slide Scanner (RSS) III

Test, Interpret, Document – The complete system for rapid tests



www.nal-vonminden.com

Rapid Slide Scanner (RSS) III

The new Rapid Slide Scanner (RSS) III offers you an optimal tool for the **easy, fast** and **objective interpretation** of all new rapid tests. The universal device can analyze **drug tests** as well as **medical assays** in various different formats. Its simple software enables a **clear analysis** and **complete documentation** of lateral flow tests.

The software recognizes the test position and form and ensures accurate result interpretation, regardless of how the test is placed in the reader. Cassette tests with barcodes can automatically be detected and analysed.

Advantages

- ✓ Easy handling
- ✓ **Free positioning** of the test
- ✓ **Easy** and **objective** evaluation in seconds
- ✓ Automatic recognition of barcoded tests
- ✓ Software in **multiple languages**
- ✓ Evaluation of up to **12 parameters** simultaneously
- ✓ Individual **result reports**
- ✓ Individual **customer support**
- ✓ **Barcode scanner** (handheld; optional) for patient data entry
- ✓ **Connection** to LIS (optional)
- ✓ Data export via Microsoft-Excel™ or CSV (optional)
- ✓ Data import via CSV (optional)
- ✓ **Remote support** and update with internet connection



The device for a huge product variety

Due to flexible test positioning capabilities and intelligent test recognition software, almost all nal von minden rapid tests can be analysed and interpreted with the RSS III. Amongst a wide variety of test formats for both drug and medical testing, tests in cup format can also be easily positioned and evaluated.

nal von minden can also offer a tool for the automatic recognition of barcoded tests for well as an even more intuitive handling of the RSS III.



Test Types:
Drug and Medical Rapid Test



Urine Cups



Barcoded tests
For a fully automated and accurate analysis

Your choice

- ✓ Drug-Screen® single dip / pipette test
- ✓ Drug-Screen® multi dip / pipette test
- ✓ Drug-Screen® Urine Cup III
- ✓ NADAL® Medical rapid tests
- ✓ Adulteration parameter
- ✓ Different matrices:
urine, saliva, whole blood / serum / plasma



Test & Interpret – 3 Steps to reliable results

Performance




Result

Patient Data

Test

Picture of Test

Result

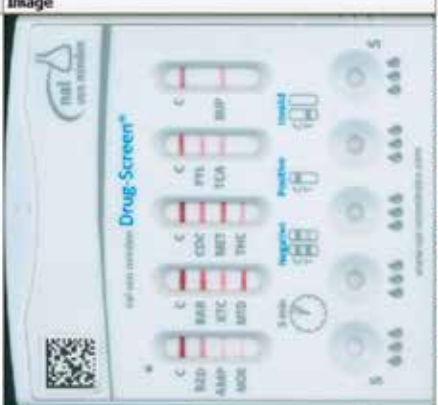
Patient	Patient-ID	Sample-I	Test	Image	Result																																							
Smith, John 27.01.2017	3132105	15321238	nal von minden 1041258A Multi 12TE 166679 Rev. 003		<table border="1"> <thead> <tr> <th>Drug shortname</th> <th>Cut off (ng/ml)</th> <th>Result</th> </tr> </thead> <tbody> <tr><td>BUP</td><td>10</td><td>negative (-)</td></tr> <tr><td>FYL</td><td>10</td><td>negative (-)</td></tr> <tr><td>TCA</td><td>1000</td><td>negative (-)</td></tr> <tr><td>COC</td><td>300</td><td>negative (-)</td></tr> <tr><td>MET</td><td>300</td><td>negative (-)</td></tr> <tr><td>THC</td><td>50</td><td>negative (-)</td></tr> <tr><td>BAR</td><td>300</td><td>negative (-)</td></tr> <tr><td>XTC</td><td>500</td><td>negative (-)</td></tr> <tr><td>MTD</td><td>300</td><td>negative (-)</td></tr> <tr><td>BZD</td><td>300</td><td>negative (-)</td></tr> <tr><td>AMP</td><td>300</td><td>negative (-)</td></tr> <tr><td>MOR</td><td>300</td><td>negative (-)</td></tr> </tbody> </table>	Drug shortname	Cut off (ng/ml)	Result	BUP	10	negative (-)	FYL	10	negative (-)	TCA	1000	negative (-)	COC	300	negative (-)	MET	300	negative (-)	THC	50	negative (-)	BAR	300	negative (-)	XTC	500	negative (-)	MTD	300	negative (-)	BZD	300	negative (-)	AMP	300	negative (-)	MOR	300	negative (-)
Drug shortname	Cut off (ng/ml)	Result																																										
BUP	10	negative (-)																																										
FYL	10	negative (-)																																										
TCA	1000	negative (-)																																										
COC	300	negative (-)																																										
MET	300	negative (-)																																										
THC	50	negative (-)																																										
BAR	300	negative (-)																																										
XTC	500	negative (-)																																										
MTD	300	negative (-)																																										
BZD	300	negative (-)																																										
AMP	300	negative (-)																																										
MOR	300	negative (-)																																										

Documentation & Export

In addition to the interpretation of rapid tests, the RSS smart software also offers a **complete documentation of patient data and results**. These are initially archived in the internal database and can be subsequently transferred via an **optional**

interface to an external data management system (e.g. LIS). The data transfer can be either **unidirectional** or **bidirectional**. If required, a clear report of every analysis can also be printed or saved in PDF format.

Clear patient management

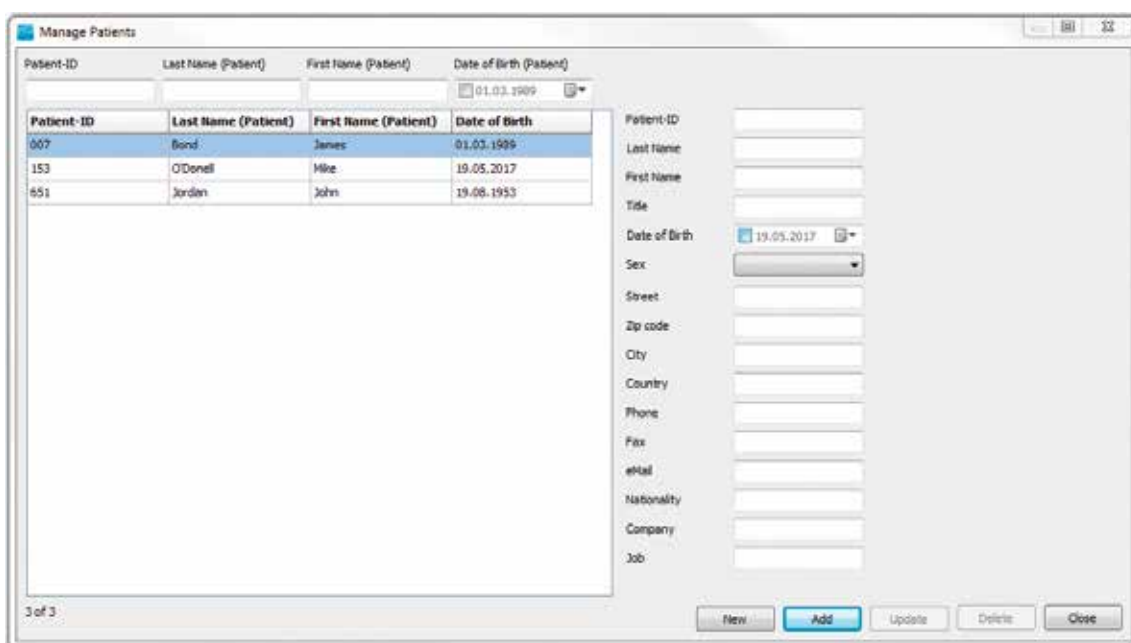


Patient-ID	Last Name (Patient)	First Name (Patient)	Date of Birth
007	Bond	Jane	01.03.1999
153	O'Donnell	Mike	19.05.2017
651	Jordan	John	19.08.1953

Patient report

With the click of a mouse you can create a full patient report with analysis data. This report is customised to your test requirements and the layout will be adapted to the standards of your laboratory or surgery.

Patient report features

- 1 Your individual logo and laboratory details
- 2 Patient's data (identification)
- 3 Complete scan – picture of the test
- 4 Result for tested substances

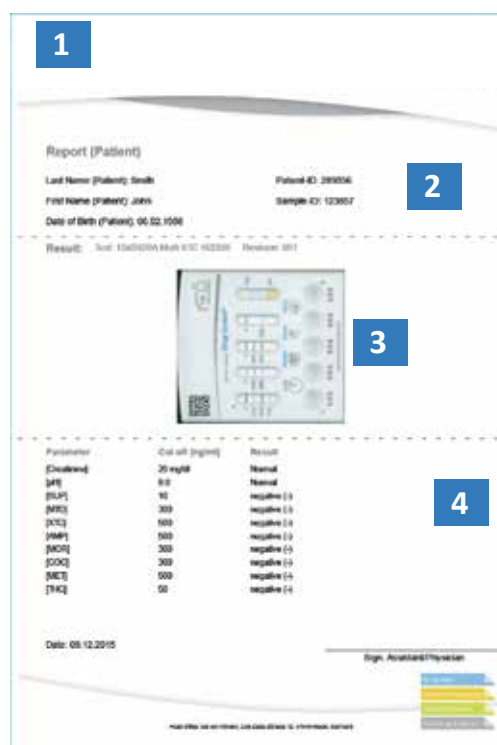


Report (Patient)

Last Name (Patient): Smith Patient-ID: 289056
 First Name (Patient): John Sample-ID: 123657
 Date of Birth (Patient): 04.02.1998

Result: Test: 13d30194-Mark-010-02200 Revision: 001

Parameter	Unit (if any)	Result
[Cholesterol]	20 mg/dL	Normal
[Hb]	g/L	Normal
[HPL]	10	negative (1)
[HPL]	300	negative (1)
[HPL]	600	negative (1)
[HPL]	900	negative (1)
[HPL]	300	negative (1)
[HPL]	300	negative (1)
[HPL]	500	negative (1)
[HPL]	00	negative (1)

Date: 08.12.2015

Sign: 

Ordering information

Article	Description	Article number
Rapid Slide Scanner (RSS) III	Analyser with drawer Power supply USB cable Calibration card Manual Installation CD	770066
Rapid Slide Scanner (RSS) III-Cup	Analyser with cup-drawer Cup base unit Power supply USB cable Calibration card Manual Installation CD	770061
Upgrade Kit	Cup-drawer Cup base unit	770063
Import/Export license	License key for the activation of the import/export function; enables export as CSV file	770010
Voucher for 5000 strip analyses	Activation key for the analysis of 5000 strips	770100
Barcode scanner	Barcode scanner (handheld) incl. mount; connection via USB cable	770015

Expand your range...

nal von minden is your specialist for drugs of abuse and medical testing in Europe, with the widest range of drugs of abuse tests in all formats.

- ✓ Tests for over 31 different drugs of abuse, including fentanyl, tramadol, spice and a wide choice of cutt-off levels.
- ✓ Production of your individual parameter combination requirements on request.
- ✓ Single and multi-tests for urine, saliva, whole blood and faeces in the following formats: dip-test, cassette test, cup test or integrated tests (incl. sample collector).
- ✓ Tests to detect sample adulteration.
- ✓ Extensive portfolio of medical rapid tests for bacteriology, infectiology, tropical medicine, parasitology and many more.
- ✓ Possibility for further laboratory analysis (ultrasensitive drugs of abuse EIA & DOA as well as medical ELISA Tests) and confirmatory analysis through our partner labs.

Technical data

Features	
Number of tests per analysis	1-2 cassette tests
Device size (770066)	296 x 230 x 77 mm
Device size (770061)	296 x 230 x 130 mm
Weight	2,9 kg
Scan procedure	Scan once
Lamp	LED light source
Current consumption	18 W (active); 5 W (passive)
Power supply	1x external power supply, CE certified
Input	100V - 240V, 50 - 60 Hz, 0.6 A
Output	24V - 0.75 A
Cable length	1500 mm
USB cable length	1500 mm
Scan size	70 x 150 mm
Technology	Color CCD Sensor
Humidity	20-80%, non condensing
Operation temperature	15-35°C
Working height	0 to 2000m above sea-level





Headquarters (Germany)

Carl-Zeiss-Strasse 12, 47445 Moers
Tel.:+49 2841 99 820-0
Fax:+49 2841 99 820-1

Friedenstrasse 32, 93053 Regensburg
Tel.:+49 941 290 10-0
Fax:+49 941 290 10-50

info@nal-vonminden.de
www.nal-vonminden.de



Belgique / België

Français:

Numéro gratuit: 0800 718 82
Fax gratuit: 0800 747 07
dept.ventes@nal-vonminden.com
www.nal-vonminden.fr

Nederlands:

Gratis Tel: 0800 780 85
Fax: +31 70 3030 775
verkoop.dept@nal-vonminden.nl
www.nal-vonminden.nl

Česká republika

Bezplatný tel.: 800 143 685
Bezplatný fax: 800 143 686
czech@nal-vonminden.com
www.nal-vonminden.cz

Danmark

Gratis nummer: 80888753
Fax: +31 7030 30775
distribution@nal-vonminden.dk
www.nal-vonminden.dk

Deutschland

Gratis-Tel.: 0800 589 2022
Gratis-Fax: 0800 589 2023
vertrieb.abt@nal-vonminden.de
www.nal-vonminden.de

España

Tel. gratuito: 900 938 315
Tel. directo: +49 941 290 10-759
ventas@nal-vonminden.es
www.nal-vonminden.es

France

Numéro vert: 0800 915 240
Fax gratuit: 0800 909 493
N° international: .. +49 941 290 10-739
Fax international: ... +49 941 290 10-50
dept.ventes@nal-vonminden.fr
www.nal-vonminden.fr

Ireland

Freephone: 1800 555 008
Freefax: 1800 555 080
sales-dept@nal-vonminden.com
www.nal-vonminden.com

Italia

Numero verde: 800 789 129
Fax gratuito: 800 789 130
vendite.dipt@nal-vonminden.com
www.nal-vonminden.it

Latinoamérica

Tel. directo: +49 941 290 10-761
ventas.latam@nvm-diagnostics.com
www.nal-vonminden.es

Luxembourg

Numéro gratuit: 800 211 16
Fax gratuit: 800 261 79
dept.ventes@nal-vonminden.com
www.nal-vonminden.fr

Nederland

Gratis Tel: 0800 022 2890
Gratis Fax: 0800 024 95 19
verkoop.dept@nal-vonminden.nl
www.nal-vonminden.nl

Norge

Free phone: 800 16 731
norway@nal-vonminden.com
www.nal-vonminden.com/no

Österreich

Gratis Tel: 0800 291 565
Gratis Fax: 0800 298 197
vertrieb.abt@nal-vonminden.de
www.nal-vonminden.de

Polska

Bezpłatny tel.: 00800 4911595
Bezpłatny fax: 00800 4911594
sprzedaz.dept@nal-vonminden.pl
www.nal-vonminden.pl

Portugal

Tel. gratuito: 800 849 230
Tel. direto: +49 941 290 10-735
vendas@nal-vonminden.pt
www.nal-vonminden.pt

Schweiz / Suisse / Svizzera

Deutsch:

Gratis-Tel.: 0800 832 164
Gratis-Fax: 0800 837 476
info@nal-vonminden.de
www.nal-vonminden.de

Français:

Numéro gratuit: 0800 564 720
Fax gratuit: 0800 837 476
dept.ventes@nal-vonminden.com
www.nal-vonminden.fr

Italiano:

Numero verde: 0800 832 2146
Fax gratuito: 0800 837 476
vendite.dipt@nal-vonminden.com
www.nal-vonminden.it

Suomi

Ilmainen Puh: 0800 918263
Ilmainen Fax: 0800 918262
myynti@nal-vonminden.com
www.nal-vonminden.com/fi

Sverige

Gratis Tel.: 020 790906
sweden@nal-vonminden.com
www.nal-vonminden.se

United Kingdom

Freephone: 0808 234 1237
Direct dial: +44 1580 23 22 88
sales-dept@nal-vonminden.co.uk
www.nal-vonminden.com

info@nal-vonminden.com
www.nal-vonminden.com